

AKVOSEGO™



BATH CHAIR



User manual

REH  **MAT**
MANUFACTURER OF MODERN ORTHOPAEDIC BRACES



CAUTION! Manufacturer takes responsibility for proper usage of the device only when the product was purchased from qualified specialist at medical store.

CAUTION! REH4MAT reserve the right to introduce technical and trading changes in the content of the instruction without warning.

Medical product first class according to the Regulation 93/42/EWG of European Parliament and Council regarding medical products and with the Act of Law of 20 May 2010 on Medical Devices (Dz. U. 2010 nr 107 poz 679)

**COMPANY WITH
QUALITY SYSTEM
CERTIFIED BY DNV GL**

= ISO 13485 =



CONTENTS

1. The purpose of user manual.....5

2. Indications for use.....5

3. General characteristics of bath chair.....6

 3.1. Standard equipment.....6

 3.2. Additional equipment.....6

4. Size table7

5. Detailed description of how to apply the lounge upholstery.....8

6. The adjustment of bath chair.....11

7. Additional equipment.....14

8. How to fix bodymap cushion on chair.....16

9. Using of chair.....18

10. Safety rules.....19

11. Maintenance and cleaning.....21



INTRODUCTION

AKVOSEGO™ bath chair is simple and innovative solution that is helpful while taking care of disabled people. The bath chair consists of solid frame that is made of aluminium and, moreover, it protects from adverse effects of water. Furthermore, frame is equipped with foam covers that protect bathtub or kids bath tub from scratching. Unique construction of frame makes it possible to adjust the tilt angle of footrest and backrest.

Furthermore, it is also possible to change distance as well as seat's tilt angle towards floor. It is highly beneficial due to the fact that product is correctly adjusted to user's needs which is very helpful while e.g. washing little user's hair.

Concerning upholstery, it is made of impermeable net. AKVOSEGO™ bath chair can be used together with BodyMap® cushion either in a room or during bath because all BodyMap® cushions are totally waterproof! The use of BodyMap® cushion makes the stabilization of patient more efficient during bath.

Its upholstery and frame are available in different colours – it depends on its size.



CAUTION! Before using the product, user is obliged to familiarize himself/herself with user manual. Remember that complying with guideline is highly important. It assures safety and extends durability as well as aesthetics of the device.



1 THE PURPOSE OF USER MANUAL

This manual contains information necessary for correct use of **AKVOSEGO™** bath chair, preparation of the device before the use, maintenance and cleaning as well as the guarantee conditions. Manual should be kept in an easy accessible place.

The manual is addressed to the parents, doctors and physicaltherapistis who take direct care of handicap children.



CAUTION! Before using the product, user is obliged to familiarize himself/herself with user manual.

Please, remember that following the rules of the manual enables safe use of the product, its extended life and esthetics.

2 INDICATIONS FOR USE

AKVOSEGO™ bath chair is created for children with infantile cerebral palsy (CP) to maintain their health and hygiene. CP depends on its stage and can occurs as an:

- Hemiplegia, characterized by only one side muscles weakness, based on muscles spasticity or flaccid paralysis.
- Diplegia usually involves lower limbs where gait learning is very difficult. Patients often use walkers or crutches.
- Bilateral hemiplegia is connected with extension position and limited motions. With age, paresis of limbs can occur, mainly of upper limbs. Trunk and head musles are weak, usually.
- Tetraplegia involves whole body: head, trunk and limbs. Patients, depending on stage, can crawl, sit or lie.




3 GENERAL CHARACTERISTICS OF BATH CHAIR

3.1 Standard equipment

1. Frame
2. Head support
3. Trunk support
4. Upholstery not absorbing the water
5. Leg support



3.2. Additional equipment

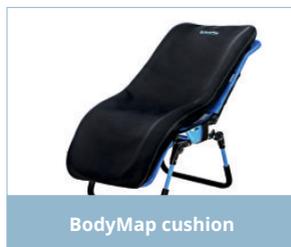



4 SIZE TABLE



Symbol	Measurement	Unit	Size 0	Size 1	Size 2	Size 3
A	Seat width	[cm]	37	42	45	50
B	Seat depth	[cm]	23-28	28-33	32-40	37-50
C	Head rest height	[cm]	16	16	19	23
C1	Back rest height	[cm]	31	40	44	50
C2	Lower leg length	[cm]	26	31	36	41
C3	Floor to seat distance	[cm]	11-44	11-44	13-49	15-54
	User weight (max)	[kg]	25	30	50	75

Weight and dimensions of the device		Size 0	Size 1	Size 2	Size 3
Product width	[cm]	42	42	45	50
After folding width	[cm]	42	42	45	50
Product length	[cm]	70	75	91	97
After folding length	[cm]	64	67	76	88
Product height	[cm]	80	85	96	113
After folding height	[cm]	28	28	29	31
Product weight	[kg]	4	4	4,5	8

Note: Dimensional tolerance in the table is +/-1 cm.



5 DETAILED DESCRIPTION OF HOW TO APPLY THE LOUNGER

5.1. Components :



- 1** Frame of bath deckchair x 1
- 2** Footrest frame
- 3** Backrest Frame
- 4** Joints
- 5** Head support x 1
- 6** Upholstery x 1



To put on the upholstery on the frame first you must :

- Press joints (G and F) at the same time adjust the angle between the backrest and the footrest (according to fig. 1),

-Insert the upper part of upholstery on the frame of the back support (fig. 2, 3),

-Push the bottom part of the upholstery onto the footrest frame (fig. 4),



fig.1



fig.2



fig.3



fig.4

Fasten all straps with velcro closures from the underside of the lounger (fig. 5, 6). Please keep in mind that the straps should cover the seat frame.

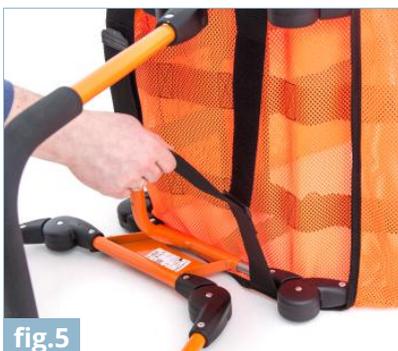


fig.5



fig.6



To mount the head stabilizer you must:

- choose the appropriate height of the stabilizer on the backrest of the lounger,
- next using straps with velcro closures fasten the stabilizer on the back of the lounger (fig. 7, 8).



-lounger with upholstery set to the usable position (fig.9).





6 THE ADJUSTMENT OF BATH CHAIR

It is necessary to use special joints so as to adjust bath chair **AKVOSEGO™**.
It is possible to adjust following parts of bath chair **AKVOSEGO™** :

BACKREST



FOOTREST





THE TILT ANGLE OF BATH CHAIR



Every standard bath chair **AKVOSEGO™** is equipped with head support, trunk stabilizing belt and feet stabilizing belts. All these parts secure the patient while using the product. Please follow these steps mentioned below to use the product:

STEP 1. Fix belt with head stabilizing pelottes at chosen height. Belt should be fastened at the back of seat by means of Velcro tapes. The distance of pelottes can be adjusted.





STEP 2. Fix trunk stabilizing belt. Belt is fastened with the aid of Velcro tapes and it can be fixed at any height. It is necessary to thread belt through grommets which are situated on either backrest or seat. Next, fasten the belt.



STEP 3. Fix feet stabilizing belts. Belts are fastened with the aid of Velcro tapes and they can be fixed at any height. It is necessary to thread belts through grommets which are situated on footrest. Next, fasten belts.





6.2. Seat depth adjustment

In order to adjust the depth of seat, there is necessary to press the buttons on both sides at the same time.



6.3. Upholster

Tension of upholster can be easily adjusted by straps with Velcros that are located under the chair.

7 ADDITIONAL EQUIPMENT

The pelvic-thigh belt ensures an adequate level of stability and safety while sitting in the seat. To attach the belt, the bottom strap with buckles must be put under the seat and then fastened.



bottom strap with buckles



Afterwards, insert the two side straps into regulators as explained below. Properly attached belt is shown in the last picture.



side straps with regulators






8 HOW TO FIX BODYMAP CUSHION ON CHAIR

All BodyMap® cushions are waterproof and highly thanks to which it is recommended to use them together with bath chair **AKVOSEGO™** so as to enhance upright standing and secure the patient while performing nursing activities.

Please follow steps mentioned below to fix BodyMap® cushion:

STEP 1. Disassemble head support, trunk stabilizing belt and feet stabilizing belts.

STEP 2. Fix BodyMap cushions with the aid of fastening set that is included in optional equipment for bath chair **AKVOSEGO™**. Each set is equipped with pelvic-thigh belt, two wide belts that stabilize the patient and three Velcro tapes that are used to place BodyMap® cushion on seat.

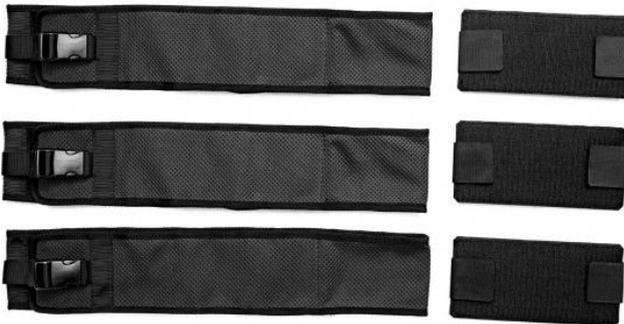



Before fixing the BodyMap® cushion on the both seat, it is necessary to assemble special Velcros in lower, medium and upper part of the seat. Self-adhesive surface of BodyMap cushion makes it possible to fix it quickly on bath seat.



STEP 3. Then fix belts stabilizing the patient. Belts are fastened by Velcro tapes and buckle. Below you can see the conect assembling process:



STEP 4. To fix BodyMap® cushion it is necessary to unfasten stabilizing belts. Place cushion on bath chair **AKVOSEGO™** in a way that its self-adhesive surface touches bath seat's upholstery. Next, place the patient on cushion and fasten stabilizing belts. Please remember that BodyMap® cushion has to be shaped according to patient's body because only then cushion correctly stabilizes the patient both in sitting. Please familiarize yourself with the guidelines concerning shaping the cushions which are included in user manual for cushions.





9 USING OF CHAIR



CAUTION! The product has to be used with compliance to guidelines included in this user manual and with trained person. Before every using, there is necessary to prepare the patient and the device!

Please follow these steps below before the use of product:

- Unfold the device,
- Place the device on even surface, without slopes,
- Make sure that all parts are serviceable, correctly mounted and stable,
- Adjust bath seat and head support to patient's measurements,
- Unfasten trunk stabilizing belt and feet stabilizing belts,
- Place the patient in bath seat and secure him/her by set of belts from standard equipment.

If product is used together with BodyMap cushion; first fix cushion on bath chair with the set of fastening and then shape it according to patient's body.

our device works correctly only when the product is adjusted according to patient's height as well as weight. Remember to take all these measurements into account while purchasing the product. Correct adjustment of bath chair is the most important thing due to the fact that only then the product is safe. It is forbidden to force child to keep specific position because he/she will feel uncomfortable and it may lead to wrong posture.



CAUTION! Befor using, remember to make sure that all belts and locking are correctly adjusted and fastened.



10 SAFETY RULES



CAUTION! AKVOSEGO™ bath chair should be used in compliance with guidelines described in instruction manual.

It is highly important to consult either doctor or physiotherapist in order to make right decision concerning the choice of chair **AKVOSEGO™** and its size. It is essential to familiarize with the guidelines described in instruction manual before using the device.

1. Before using the product, it is essential to make sure that:

- make sure that there are no damages or different faults which could endanger patient's life. Make sure that all parts are operating correctly and are properly mounted. Moreover, make sure that there are no cracks as well as deformities,
 - make sure that all screws, nuts and other threaded, glued and sewn parts are firmly tightened;
 - make sure that buckles are operating correctly;
 - always use stabilizing belts: Velcro tapes from these belts has to overlap min. 8cm;
 - adjust the product to patient's measurements as well as needs;
 - secure the patient with the aid of head support, trunk stabilizing belt and feet stabilizing belts. It is necessary due to the fact that it protects the patient against falling out of the device.
2. It is forbidden to use bath chair for different purposes than it is intended to.
 3. It is forbidden to use bath chair if it is destroyed or any part is missing.
 4. It is forbidden to leave the patient unattended.
 5. It is forbidden to use the product without trained person or medical staff.
 6. It is necessary to fold and secure bath child seat during transportation in order to avoid any possible damages. Manufacturer doesn't take responsibility for damages which appeared due to improper preparation of the device during transportation.



7. It is forbidden to place and use the product close to fire or high temperatures. Bath child seat cannot be stored as well as used in rooms with high humidity.
8. Before using the chair remove all paper and plastic package .The package is not a toy. Keep away from children and animals.

CAUTION! The product contains small parts which children may swallow.



CAUTION! It is forbidden to make any changes concerning seat's structure due to the fact that it is dangerous and it results in losing warranty.

CAUTION! It is forbidden to exceed permissible weight load while using the product. All information concerning weight load is described on nameplate. Additionally, you can familiarize yourself with the table which you can find in this user manual on page 7.

CAUTION! **AKVOSEGO™** chair should be used in compliance with guidelines described in instruction manual.



11 MAINTENANCE AND CLEANING

AKVOSEGO™ bath chair should be kept clean and used in compliance with the manufacturer's recommendations. The device should be kept in clean, dry and ventilated room. Maintenance as well as cleaning have to be performed at least once a month. Regular cleaning and maintenance of bath chair are indispensable to maintain its correct functionality as well as safety.

It is highly advisable to remember about proper maintenance in order to extend product's aesthetics. So as to do that, it is essential to comply with these guidelines:

- It is possible to disassemble upholstery and clean it with the aid of washing machine, Maximum temperature is 60 degrees;
- It is forbidden to clean upholstery by means of solvents and caustic chemicals;
- In case of disinfection of upholstery, use products which are intended for it;
- Do not chlorinate;
- Do not iron;
- Dry far from sources of fire.

CAUTION! Manufacturer takes no responsibility for damages caused by use of wrong clearing agent.



CAUTION! Manufacturer takes responsibility only for latent defect or damages which appeared despite the fact that user complied with the guidelines.

CAUTION! If any damages appear, it is necessary to contact authorised salesman who sold you the product.





ORTHOPAEDIC DEVICES MANUFACTURER



36-060 Głogów Małopolski, ul. Piaski 47, POLAND

Factory of Orthopaedic Devices:
ul.Truskawkowa 17, Widna Góra, Jarosław, POLAND
tel. 16 621 42 20, 16 621 41 35
fax 16 621 41 13
www.reh4mat.com | www.ortezy.pl
e-mail: biuro@reh4mat.com

Date of issue: 13.04.2015

The last update: 20.02.2020



REH4MAT reserve the right to introduce technical and trading changes
in the content of the instruction without warning.