



MOUNTING INSTRUCTIONS FOR FLEXPOINT® ACCESSORIES  
ZEBRA INVENTO, DALMATIAN INVENTO





**FlexPoint®** system consists of many elements which provide the correct and safe stabilization of the trunk, pelvis and lower limbs. Due to the universal attachment methods, it is ideal for the vast majority of strollers and car seats offered on the market of medical devices. The selected products of the **FlexPoint®** are equipped with additional elements allowing easier fastening of belts to people with hand disfunctions.

Depending on individual needs, the user can choose a classic solution or an advanced dynamic product. **FlexPoint®** is a system recommended especially for people with spasticity. The dynamic design of the products is based on soft, elastic and stretchable relieving elements. This guarantees the highest comfort of use. At the same time, it significantly reduces the risk of bodily injury which might be caused by, among others, the excessive pressure from inflexible belts.

The correct sitting position supported by the **FlexPoint®** means safety and comfort during work, therapy and recreation throughout the day.

## Why FlexPoint®?



### Versatility

Individual elements can be successfully used in wheelchairs and other seats used by users.



### Vertical dynamics

A resilient material more effectively distributes pressure than standard solutions for user stabilization.



### Horizontal dynamics

Products with this function are dedicated to people with increased muscle tone. They significantly reduce the amount and strength of spastic seizures and increase the comfort of use.



### Grab and Pull system™

A special system used in our vests and hip belts. The ergonomic D-shaped handle makes it easy for the user to quickly and comfortably fit the product. The system is also designed for people with hand dysfunctions.



### Skin friendly

The materials used do not cause irritation and are safe for the user.



**Before mounting the FlexPoint® product, please read the user manual  
"Postural Dynamic Support Products FlexPoint®"!**



### **FP-01 DYNAMIC 4-POINT CHEST VEST**

FP-01/WZ SLIM VEST WITH ZIPPER  
FP-01/WB SLIM VEST WITHOUT ZIPPER  
FP-01/SZ STANDARD VEST WITH ZIPPER  
FP-01/SB STANDARD VEST WITHOUT ZIPPER



### **FP-02 DYNAMIC H HARNESS**



### **FP-03 CLASSIC H HARNESS**



### **FP-06 DYNAMIC 4-POINT CHEST BELTS**

## **MOUNTING PLACES**










## MOUNTING OF THE UPPER STRAPS OF THE VESTS

- i** Unfasten the buckles on the upper straps of the vest, remove the mounting triangles, but leave the regulators.

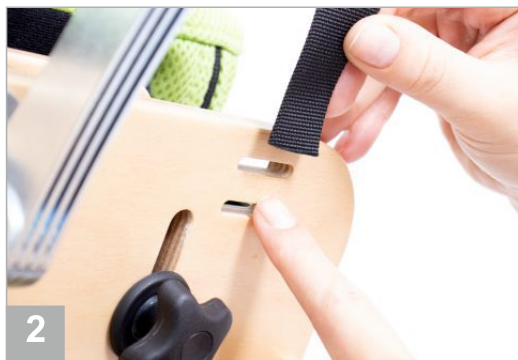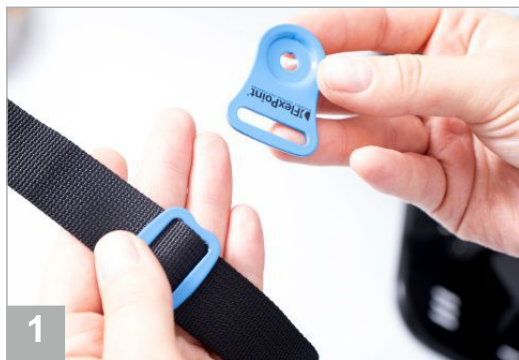

- i** Mount the upper straps of the vest in the place marked with the symbol **A**, indicated on page 4 of the manual.

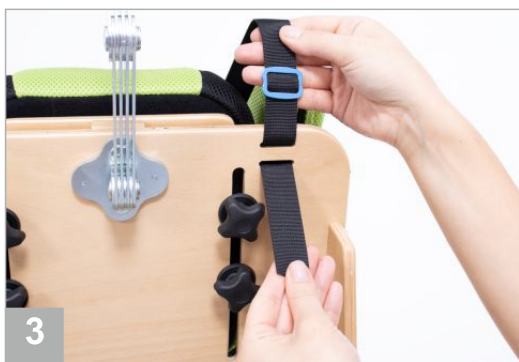





## MOUNTING OF THE LOWER STRAPS OF THE VESTS

**i** Mount the lower straps of the vest in the place marked with the symbol **B**, indicated on page **4** of the manual.

Before mounting the straps, remove the mounting triangles at their ends, but leave the regulators.

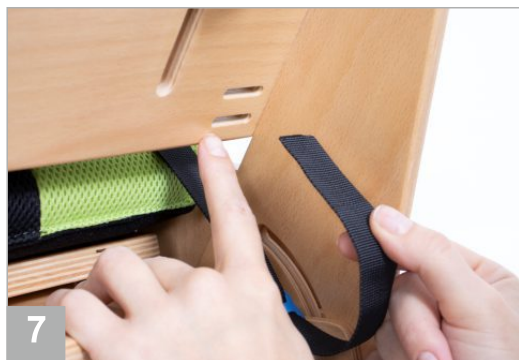

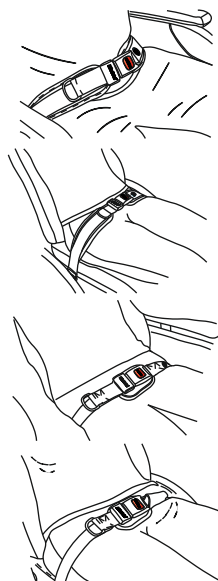

**FP-10 DYNAMIC 2-POINT BELT WITH PELVIC STABILIZATION AND ABDOMINAL RELIEF FUNCTION**

**FP-12 DYNAMIC 2-POINT PELVIC BELT**


**FP-14 2-POINT PELVIC BELT**

**FP-15 2-POINT PELVIC BELT WITH FASTENING SUPPORT**


## **MOUNTING PLACES**



The place for mounting belts is on the bottom plate of the seat.

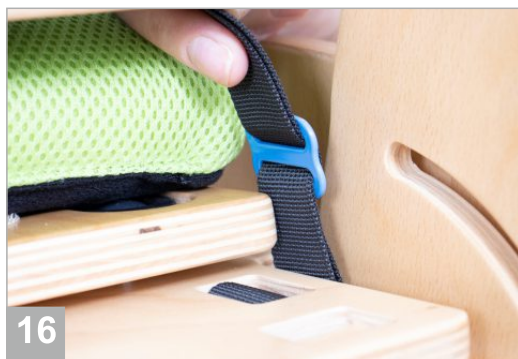



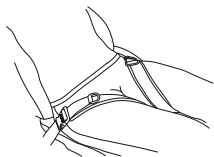

## FP-20 THIGH ABDUCTION BELTS

### REQUIRED MOUNTING ELEMENTS

#### WAY 1




ref. FP-EM-6



#### MOUNTING OF THE REAR STRAPS

Before mounting the thigh abduction belts, remove the mounting triangles at their ends. When mounting the rear straps, follow the mounting instructions for the basic pelvic stabilization straps (see pt. **Mounting places, page 7**).

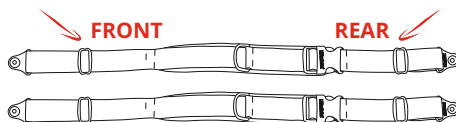

#### MOUNTING OF THE FRONT STRAPS


Position the seat of the chair so that there is an easy access to the lower surface of the seat before mounting the straps.

The recommended location of the hole for attaching the mounting triangle should be made 2.5 cm from the side edge of the seat board and 2.5 cm from the front edge.


Repeat on the other side of the seat.




17

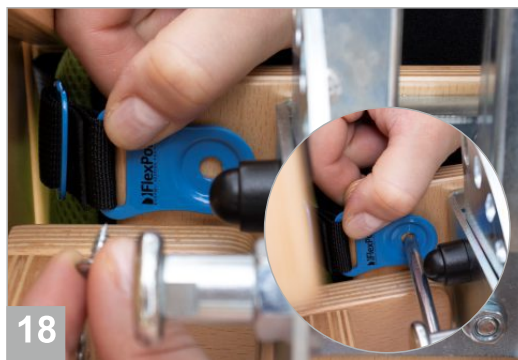

18

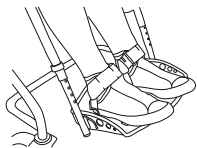


## FP-22 DYNAMIC ANKLE STABILIZERS

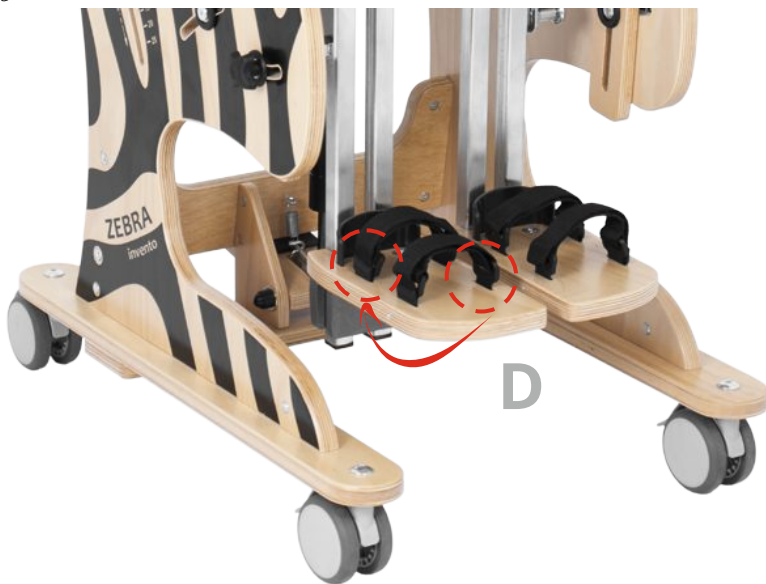




**i** Mount the Dynamic ankle stabilizers in the place marked with the symbol **D**.



19

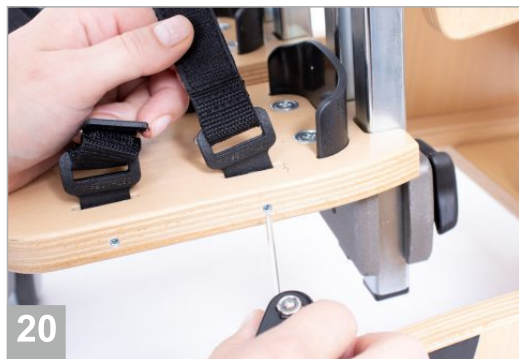

20

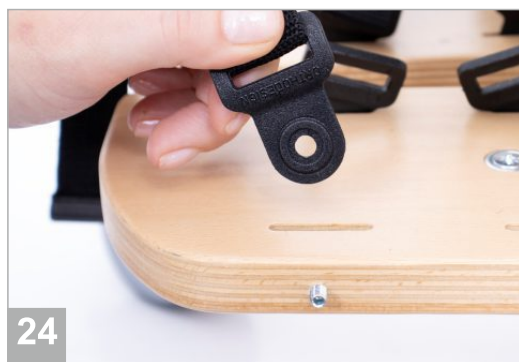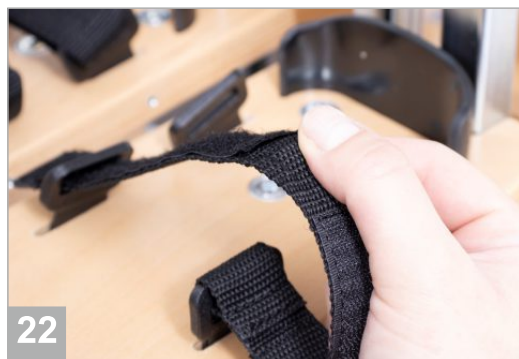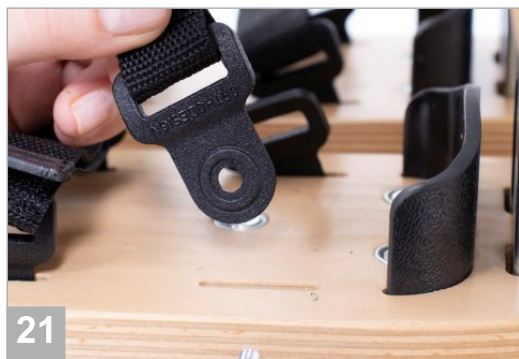



**i** Należy zabezpieczyć odkręcone śruby wkręcając je w płytę stopki (**zdj. 27**) w określonych miejscach tak, aby nie wystawały i nie stwarzały dodatkowego zagrożenia zahaczenia czy poślizgnięcia.













[flex-point.pl](https://flex-point.pl)