



TABLE
KIDOO™



USER MANUAL





Manufacturer takes responsibility for proper usage of the device only when the product was purchased from qualified **Akces-MED Ltd.** employee or at specialized medical store!

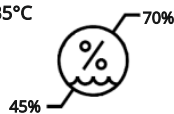
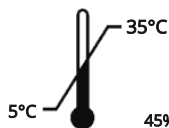
Akces-MED Ltd. reserve the right to introduce technical and trading changes in the content of the instruction without warning.



Product marked with the CE sign is Class I medical device, of rule 1, consistent with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices.



Akces-Med Ltd. Company introduced quality management system ISO 13485 regarding production, sales and service. The system is certified by Det Norske Veritas.



CONTENTS

1. Purpose of the user manual.....	5
2. Indication and contrindications for use.....	5
3. General characteristic of the table.....	6
4. Table size.....	6
5. Tray assembly.....	7
6. How to use the table.....	15
6.1 Adjusting the height of the table.....	16
6.2 Adjusting the tilt angle of the table.....	16
7. Safety rules.....	17
8. Terms of warranty.....	17
9. Labelling.....	18
9.1 Symbols.....	18
9.2 Sample data plate.....	18



INTRODUCTION

The **KIDOO™ table** is a fantastic addition to children and the youth therapy. It is a great help in everyday activities such as rehabilitation but also during play, study or having meals. An eye-catching wooden design is an ideal complement to a **KIDOO™ chair**.

The universal character of **KIDOO™ table** makes it perfect for other chairs as well as an independent product. An ergonomic shape of the tray, an adjustable height and the possibility to adjust the tabletop tilt increases the practicality and comfort while using the table. All of the above make the **KIDOO™ table** an irreplaceable element during the therapy of your child.

Remember to read this user manual carefully. You will find not only useful information but also guidelines safe and effective use of the product.

If you have any questions or remarks, do not hesitate to contact with us:

Medical adviser: mobile +48 516 183 883

Akces-MED Ltd.: ph. +48 17 864 04 77

e-mail: export@akces-med.com

Thank you for purchasing our product!



Before using the product, user is obligated to familiarize himself/herself with user manual. Remember that complying with guidelines is highly important. It assures safety and extends durability as well as aesthetics of the device.



1. THE PURPOSE OF USER MANUAL



The user manual contains basic information necessary to properly:

- prepare the table before use,
 - use the table,
- maintain and clean the table.

User manual should be kept in easily accessible place. It is designed for people who look after disabled children, and for both doctors as well as physiotherapists

who operate the device. Before using the product, user is obliged to familiarize himself/herself with user manual.

Remember that complying with guidelines is highly important. It assures safety and extends durability as well as aesthetics of the device.

2. INDICATIONS AND CONTRINDICATIONS FOR USE

The KIDOO™ table can be used in such situations as:

- Diagnosing the user – the possibility of adjusting the **KIDOO™ table** to the user's individual needs helps to analyze the clinical picture of the user, their behavior and reflexes while sitting on a bed, in a car seat, chair or wheelchair.
- Therapy monitoring during the treatment process – the daily use of the **KIDOO™ table** allows you to monitor the user's behavior during the rehabilitation process in the same, comparable conditions. It helps to choose appropriate and targeted methods to obtain the best possible results of the therapy.
- Soothing or compensating the effects of injury or handicap – the **KIDOO™ table** can be successfully used in various situations during everyday therapy, recreation and work and, at the same time, it improves the quality of everyday life. Having meals or doing simple manual activities with the use of the **KIDOO™ table** becomes much easier. The large surface of the table top makes it possible have all the necessary items within the reach of the user.

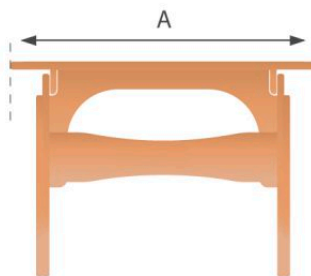
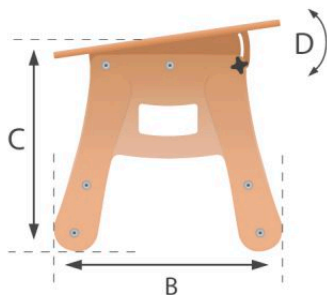


3. GENERAL CONSTRUCTION OF THE TABLE

- ① Wide board
- ② Frame with adjustable height



4. TABLE SIZE



Symbol	Dimension	Unit	Measurements			
			KIDOO™ 1	KIDOO™ 2	KIDOO™ 3	KIDOO™ 4
A	width	[cm]	65	70	75	80
B	depth	[cm]	43	48	54	58
C	height	[cm]	39 - 50	47 - 64	51 - 71	55 - 79
D	tilt adjustment range	[°]	0 - 12	0 - 12	0 - 12	0 - 12
	weight	[kg]	12,5	13,5	13,5	14,5
	max. load	[kg]	30	30	30	30



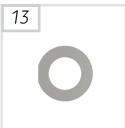
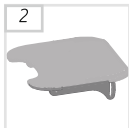





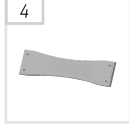








NOTE: Measurement tolerance range is +/- 1 cm / ½ in.

5. TRAY ASSEMBLY



Table of contents

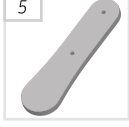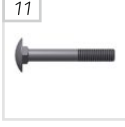
1. List of elements
2. Frame assembly
3. Hangers assembly
4. Table top assembly
5. Bracket assembly
6. Legs assembly

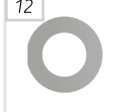1. List of elements


 <p>1 (2 pcs.)</p>	 <p>7 (2 pcs.)</p>	 <p>13 (6 pcs.)</p>
 <p>2 (1 pcs.)</p>	 <p>8 (8 pcs.)</p>	 <p>14 (10 pcs.)</p>
 <p>3 (1 pc.)</p>	 <p>9 (8 pcs.)</p>	 <p>15 (10 pcs.)</p>
 <p>4 (1 pcs.)</p>	 <p>10 (8 pcs.)</p>	 <p>16 (2 pcs.)</p>
 <p>5 (4 pcs.)</p>	 <p>11 (2 pcs.)</p>	 <p>17 (2 pcs.)</p>
 <p>6 (2 pcs.)</p>	 <p>12 (2 pcs.)</p>	 <p>18 (1 pcs.)</p>

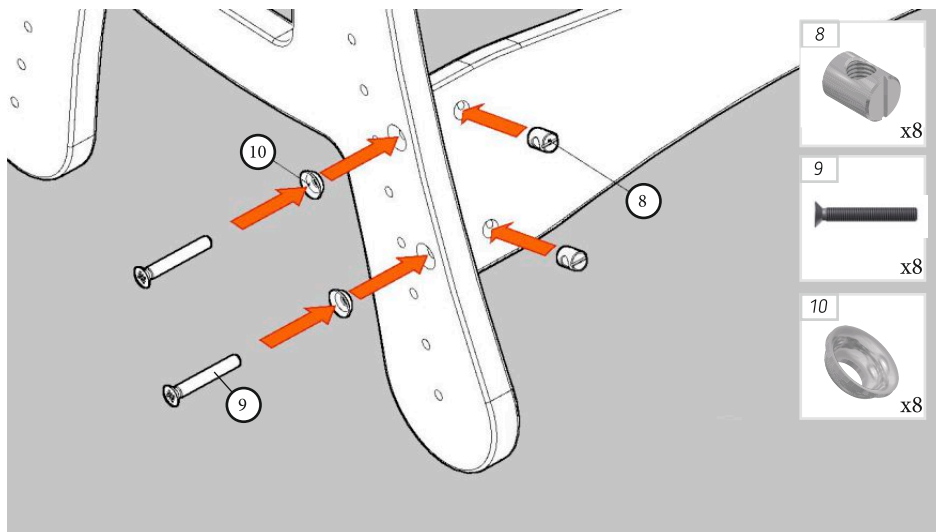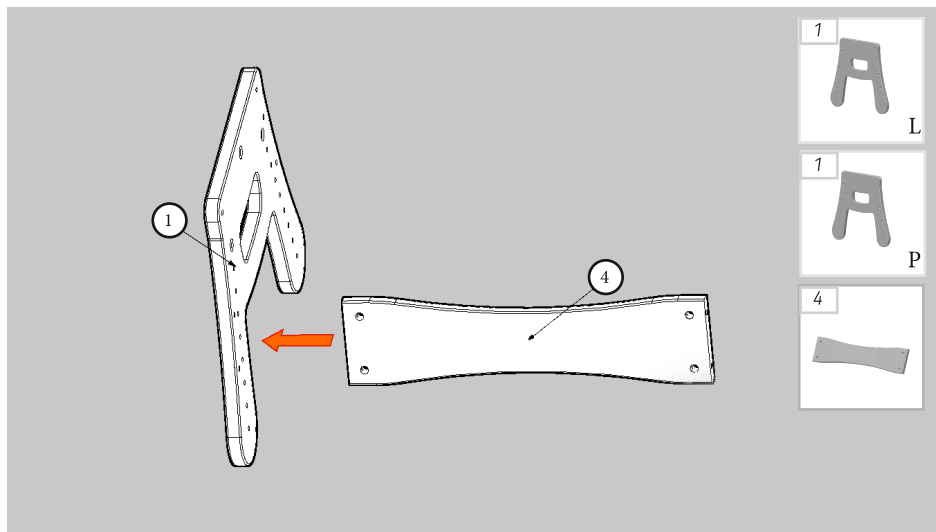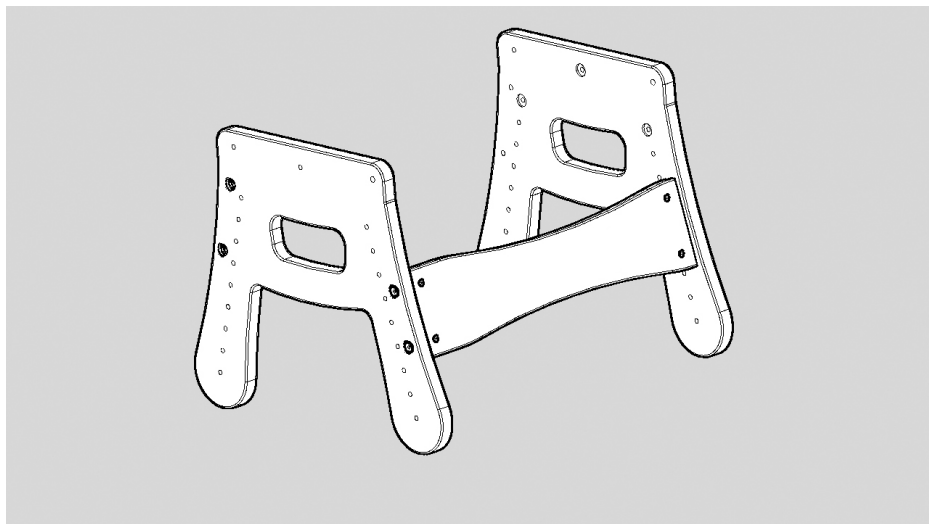
2. Frame assembly






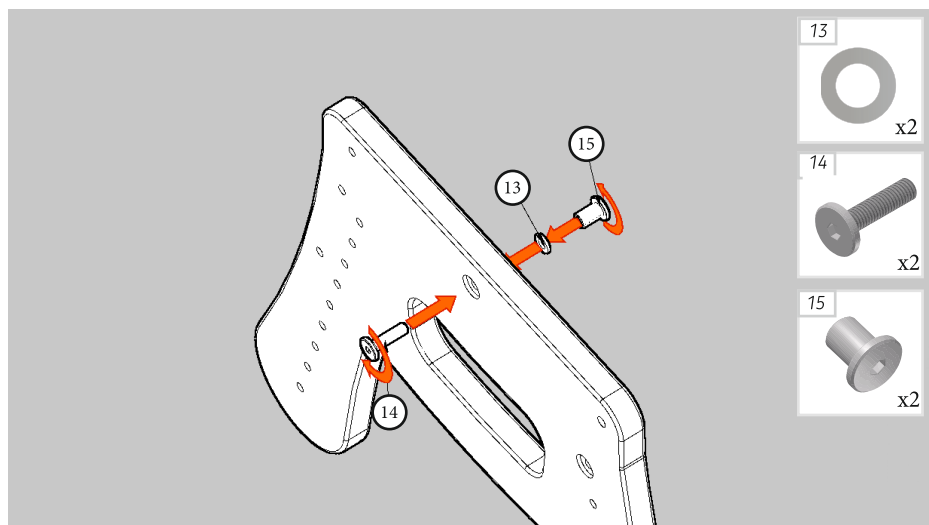
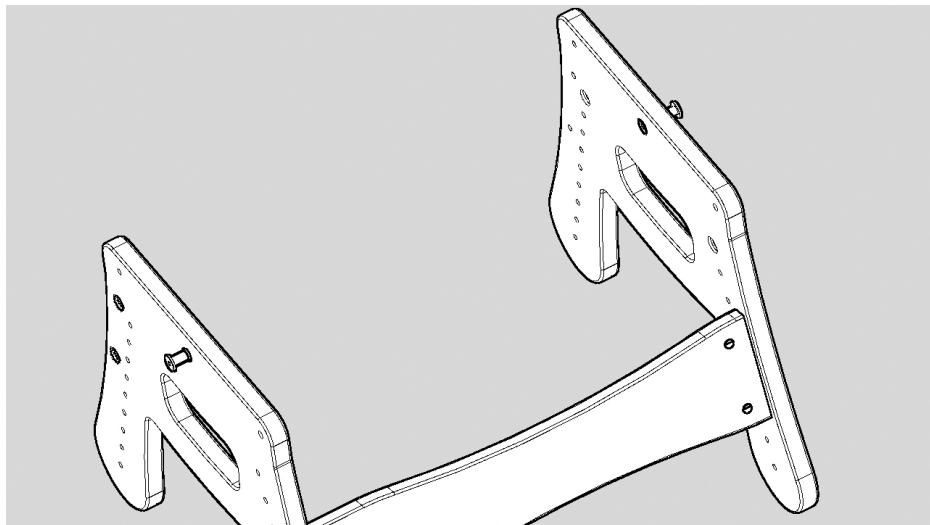
3. Hangers assmebly







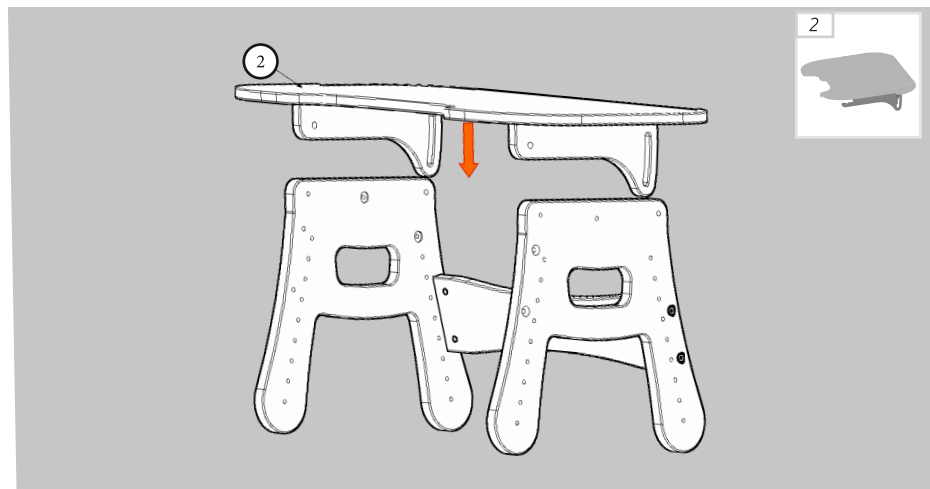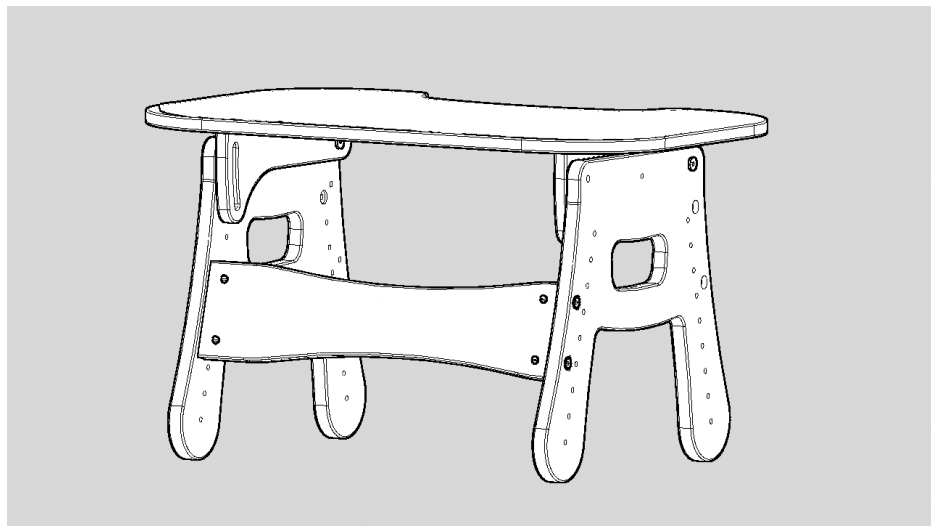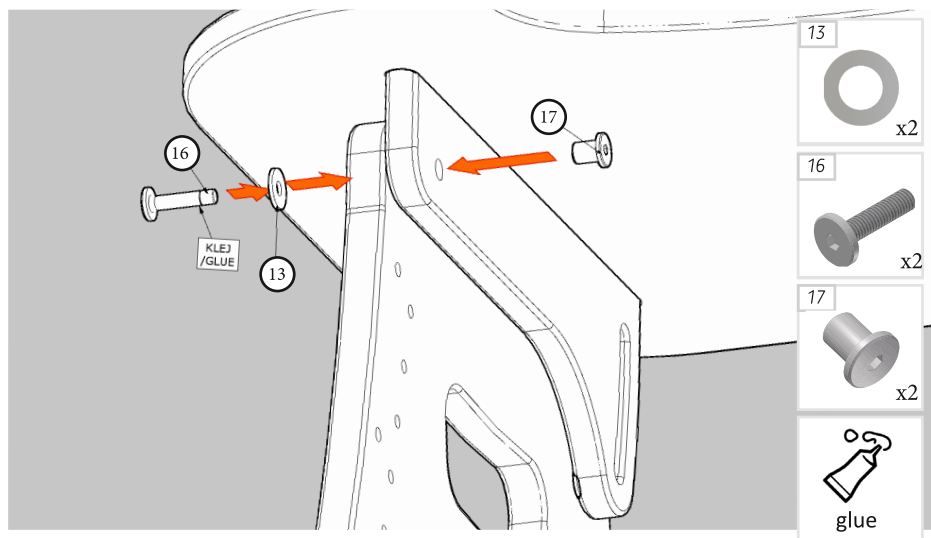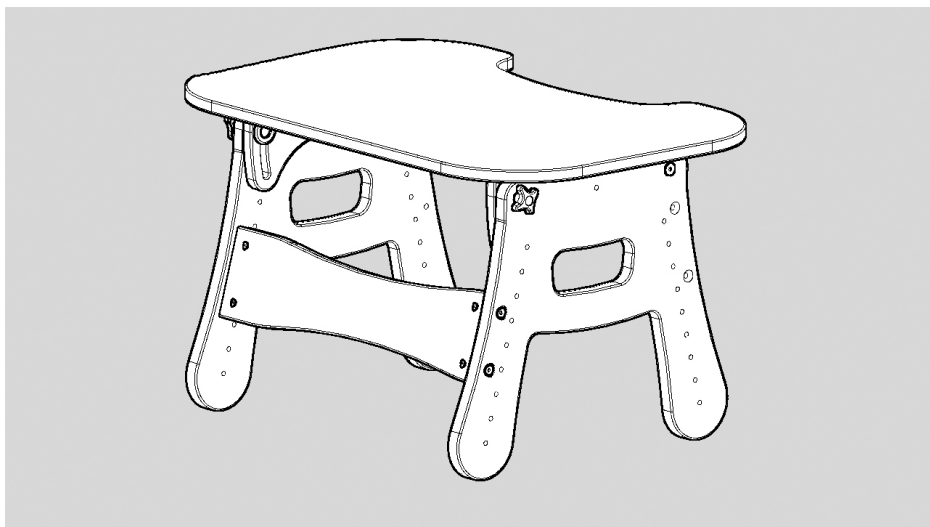
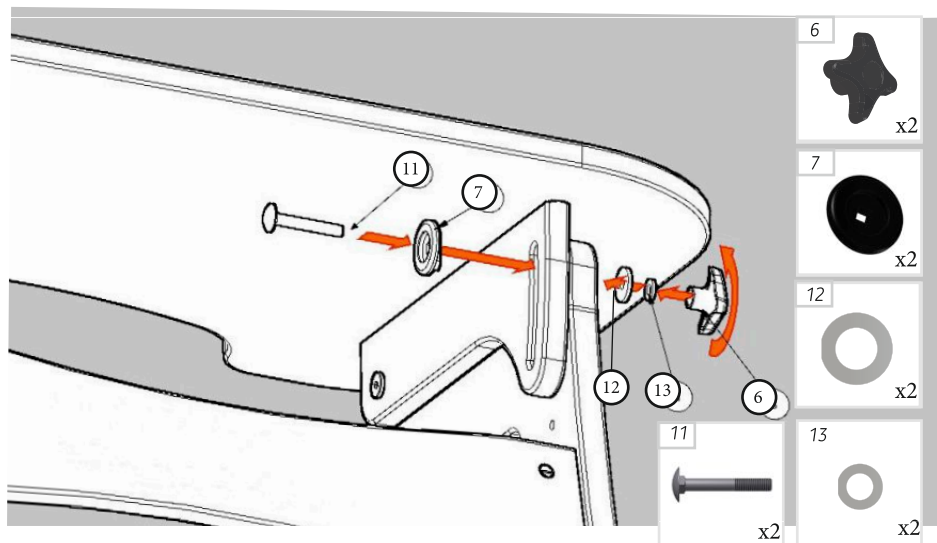
4. Table top assembly









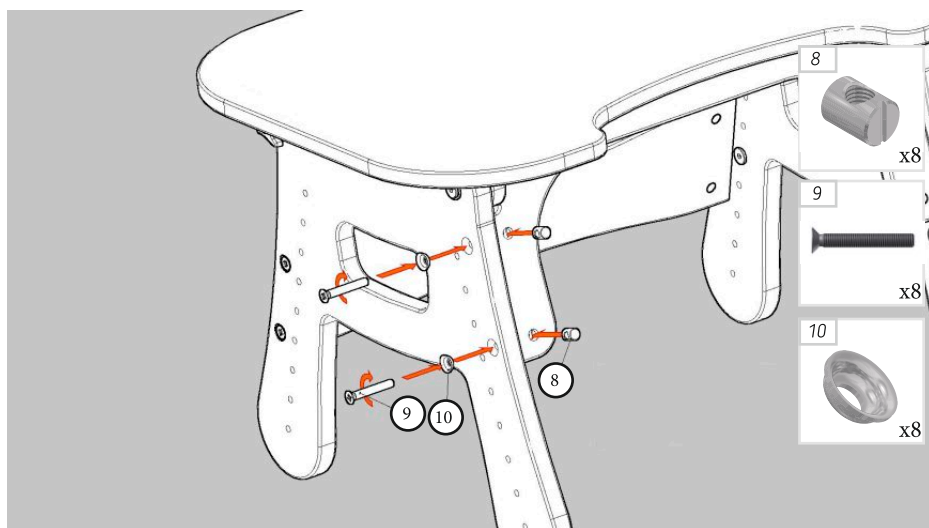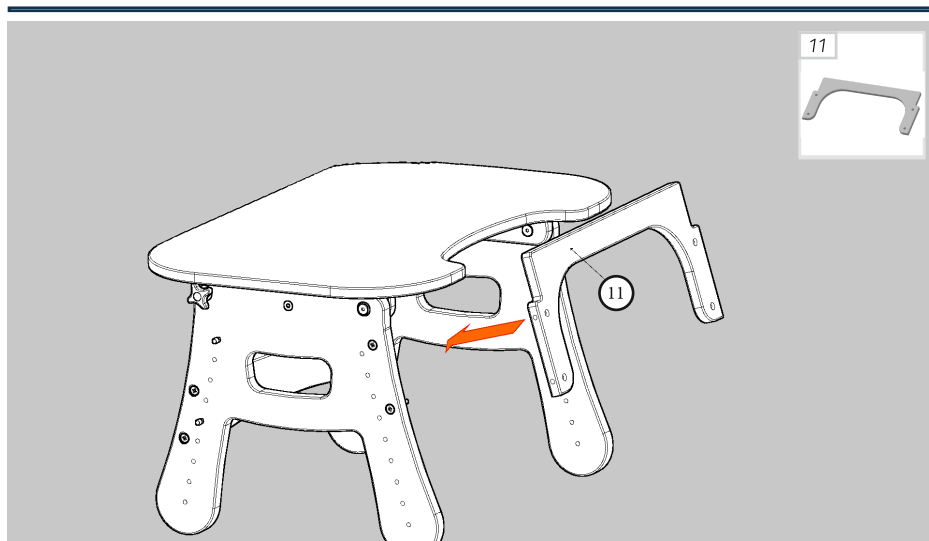5. Bracket assembly



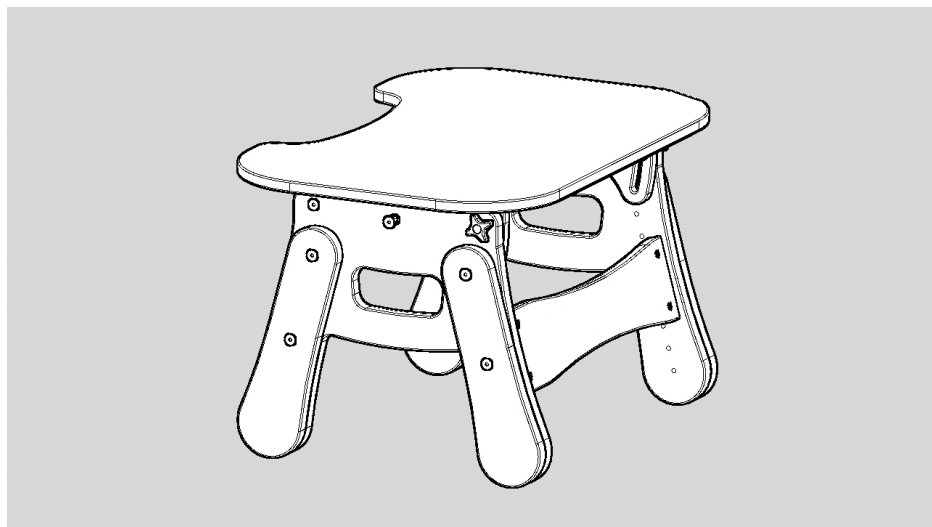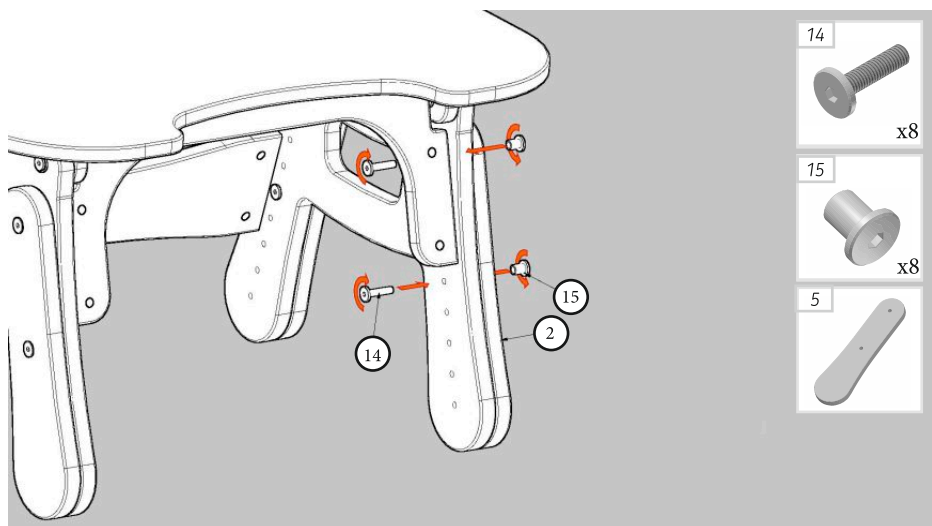6. Legs assembly



6. HOW TO USE THE TABLE

KEY



Info icon - appears in the manual when you need to read the additional information.



Info icon - appears in the manual when you need to use the tools indicated in the assembly description.



Warning icon - appears in the manual when you need to take special precautions.

6.1 ADJUSTING THE HEIGHT OF THE TABLE



To adjust the height of the table you should:

1. Unscrew the legs (**fig. 1**) using hex keys included in the kit.
2. Adjust the required height and tighten the screws (**fig. 2**).



hex key 4

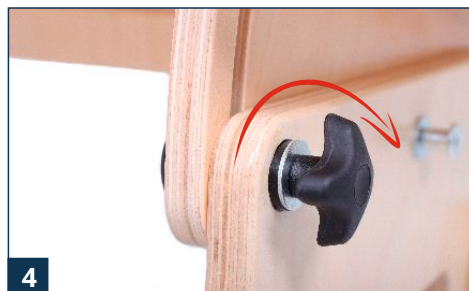


6.2 ADJUSTING THE TILT ANGLE OF THE TABLE



To adjust the tilt angle of the table you should:

1. Loosen the knobs (**fig. 3**) of the both sides of the table.
2. Adjust the required angle and tighten the knobs (**fig. 4**).



Before using the **KIDOO™ table**, make sure that all the knobs and screws of the adjustable elements have been tightened!



7. SAFETY RULES



KIDOO™ table has to be used in compliance with guidelines in this user manual.
Before every use, it is necessary to prepare the user and the device!

1. The product has to be used with compliance to guidelines included in this user manual!
Before every using, there is necessary to prepare the user and the device!
2. Before using the product, it is essential to make sure that there are no damages or different faults which could endanger user's life. Make sure that all parts are operating correctly and are properly mounted. Moreover, make sure that there are no cracks as well as deformities.
3. It is forbidden to use table for different purposes than it is intended to.
4. The table top should be cleaned with soft cloth and publicly available, mild cleaner.
5. Pay your attention during the board tilting. Please do not place the fingers between the board's and table's frame during adjustment.
6. You can disinfect the table top and metal parts of the table with a generally available alcohol-based high-concentrated disinfectant. When choosing an alcohol-based disinfectant, check on a small, invisible surface whether the product will not discolor or damage the disinfected surface.



Incorrect product adjustment can lead to such side effects.

**The product contains small parts which can be swallowed by children.
It is forbidden to make any changes in the construction of the table, as it is dangerous for the user as well as will void the warranty.**

This device can be only used inside the buildings.

The manufacturer takes full responsibility for the proper functioning of the table only when the user strictly follows the user manual.

In case of any damages, it is essential to contact the person who sold you the product.

Serious incidents involving this device should be reported both to manufacturer and to the competent authority in your country.

8. TERMS OF WARRANTY

- **Akces-MED Ltd.** sells products free from manufacturing defects or faults.
- Manufacturer gives the purchaser 12-month guarantee from the date of purchase. All defects revealed during warranty period will be repaired free of charge.
- All defects which appear after warranty period can be repaired only when the user will pay for it.
- If product is damaged, please contact our dealer or **Akces-MED Ltd.** customer service (ph. +48 17 864 04 77, e-mail: export@akces-med.com).



TECHNICAL INSPECTION









Technical inspections should be performed regularly. Before servicing, the product should be thoroughly cleaned to reveal any hidden damage.

Frequency	What should be checked? What should be done?		Remarks
Before each use.	General condition of the product.	Check: <ul style="list-style-type: none">the entire product for any visible damage, cracks or mechanical defects,,if the product complete,if all bolts, screws, knobs, nuts and other elements of the construction are complete and tightened..	In case of any damage or defects, discontinue the use of the product and contact your Supplier/Distributor or the Manufacturer.
	Fastening of the construction elements.	Check all the mechanisms of the product and its moving parts. Check in particular: <ul style="list-style-type: none">if the seat is properly mounted (if applicable),if the wheels are properly mounted (if applicable),if the elements securing the user are properly mounted and if they work well (if applicable),if all the moving parts are complete, work properly, do not jam,if the adjustment mechanisms of the following elements work properly (if applicable): headrest, backrest, seat, abduction block, armrests, footrests.	In case of any damage or defects, discontinue the use of the product and contact your Supplier/Distributor or the Manufacturer.
Once a week.	Product cleaning.	If the frame and upholstery get dirty (dusty, muddy, etc.) during everyday use, wipe them with a damp cloth without detergents. Remove dirt (e.g. hair, food remains, etc.) from the moving parts of the product.	The product should be cleaned when necessary, but at least once a week. Do not use any chlorine based cleaners or methylated spirit. Before using the product, check if the upholstery is dry.
Once a month.	Product construction connectors (welding, threading, soldering)	Check: <ul style="list-style-type: none">the condition of the product frame in the places of joining (welds, threads, soldering).	In case of any damage or defects, discontinue the use of the product and contact your Supplier/Distributor or the Manufacturer.
Once every 6 months.	Construction moving parts.	Check: <ul style="list-style-type: none">if the moving parts of the construction have not been damaged. Maintain them using a proper cleaning agent.	In case of any damage or defects, discontinue the use of the product and contact your Supplier/Distributor or the Manufacturer.
	Product labeling.	Check: <ul style="list-style-type: none">if the marking and labels are readable.	In case of any damage or defects, discontinue the use of the product and contact your Supplier/Distributor or the Manufacturer.
After the life time of the product finishes.	Detailed product check.	Check the product.	The manufacturer specifies the life time of the product, which is given in the user manual. After the given period, the user may use the product only if it is sent back to the manufacturer to be checked first. After that, the manufacturer will re-define the life time of the product and the date of the next check.


9. LABELLING

9.1 SYMBOLS

	Declaration that the product meets the essential requirements
	Catalog number
	Serial number
	Please read the manual instructions before use
	You need to take special precautions
	Manufacturer
	Production date
	The product does not contain latex
	Store in a dry place

	The table is intended for indoor use only
	Prohibition to leave the user without care during use of the product
	You need to read the additional information
	You need to use the tools indicated in the assembly description
	Careful! Delicate content
	Protect from sunlight
	Storage temperature limitation
	Relative humidity storage

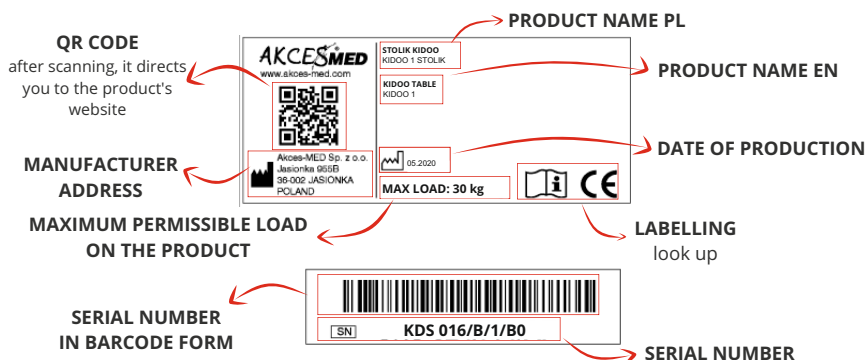9.2 SAMPLE DATA PLATE




MANUFACTURER



Akces-MED Ltd.

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Jasionka 955B
36-002 Jasionka
tel. +48 17 864 04 77
e-mail: export@akces-med.com
BDO: 000031619



Authorised Dealer



akces-med.com

Date of issue: 14.09.2018
The last update: 24.05.2021
©Sławomir Wroński 2021