



SPECIAL NEEDS STROLLER
KUKINI™



USER MANUAL





Manufacturer takes responsibility for proper usage of the device only when wyłącznie the product was purchased from qualified **Akces-MED Ltd.** employee or at specialized medical store!



Akces-MED. Ltd. reserve the right to introduce technical and trading changes in the content of the instruction without warning.



Product marked with the CE sign is Class I medical device, of rule 1, consistent with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices.



Akces-MED Ltd. introduced quality management system ISO 13485 regarding production, sales and service. The system is certified by Det Norske Veritas.

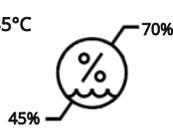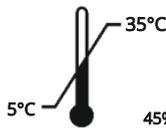

CONTENTS

1. The purpose of user manual.....	5
2. Indications and contrindications for use.....	5
3. General characteristic of the stroller.....	6
3.1 Standard equipment.....	6
3.2 Accessories.....	7
4. Size table.....	9
5. How to use the stroller.....	10
5.1 Unfolding the stroller.....	11
5.2 Folding the stroller.....	12
5.3 Footrest.....	13
5.4 Removing the wheels.....	14
5.5 5-points safety belts.....	15
5.6 Adjustable backrest.....	18
5.7 Push handle adjustment.....	18
5.8 Central brake / hand brake.....	19
5.9 Adjustment of the springs.....	19
6. Accessories.....	20
6.1 KKN_134 Narrowing pads 6 cm / KKN_137 Narrowing pads 10 cm.....	20
6.2 KKN_147 5 points feet stabilizing belts.....	21
6.3 KKN_160 Octagon™ headrest.....	22
6.4 KKN_161 Foot and ankle stabilizing sandals / KKN_162 Dynamic ankle stabilizers (FP-22).....	23
6.5 KKN_405 Folding canopy.....	24
6.6 KKN_408 Rain cover / KKN_409 Mosquito net.....	26
6.7 KKN_411 Cup holder / KKN_452 Phone holder.....	28
6.8 KKN_415 Wheels cover.....	28
6.9 KKN_445 Footmuff.....	29
6.10 KKN_505 Under seat storage basket.....	29
6.11 KKN_506 Stroller cover'n carry bag.....	30
6.12 KKN_630 Bicycle tow bar with a flag.....	31
6.13 KKN_626 EVO module.....	34
7. The use of the stroller.....	35
7.1 Preparation of the user.....	35
7.2 Preparation of the stroller.....	36
8. Safety rules.....	37
9. Terms of warranty.....	39
10. Maintenance and cleaning.....	39
11. Labelling.....	43
11.1 Symbols.....	43
11.2 Sample data plate.....	43

INTRODUCTION

To meet the expectations of our customers, the Akces-MED Ltd. team created a new special needs stroller **KUKINI™**. It's really light so it is perfect for both long walks and running trainings. Aluminium frame of the stroller with reflective elements makes the stroller safe and comfortable for your child while exploring the city or forest areas.

KUKINI™ is equipped with a rear shock absorbing system and 16-inch inflatable spoked wheels, which make riding on uneven terrain a pleasure for both a child and caregiver.

The child's safety and correct sitting position are guaranteed by a 5-point safety belts with a magnetic fastener. The comfortable head support is provided by the anatomical Octagon™ headrest, which can be mounted at different heights. An adjustable backrest allows you to set the optimal position for your child while riding. The wheel quick release system together with the compact size of the folded stroller make it easy to store or transport. Durable and comfortable upholstery has a practical pocket to hold your phone, wallet or keys.

In order to protect the child from various weather conditions, the special needs stroller **KIKINI™** can be equipped with an adjustable folding canopy to provide additional protection from wind, rain or sun.

Remember to read this user manual carefully. You will find not only useful information but also guidelines safe and effective use of the product.

If you have any questions or remarks, do not hesitate to contact with us:

Medical adviser: mobile +48 516 183 883

Akces-MED Ltd.: ph. +48 17 864 04 70

e-mail: export@akces-med.com



Before using the product, user is obliged to familiarize himself/herself with user manual.

Remember that complying with guidelines is highly important. It assures safety and extends durability as well as aesthetics of the device.





1. THE PURPOSE OF USER MANUAL



The instruction manual contains basic information necessary to properly prepare the stroller before use, use the stroller, maintain and clean the stroller.

This user manual should be kept in an easily accessible place. The manual is intended for people who look after disabled users, as well as for doctors and physiotherapists handling the stroller.

Before using the stroller for the first time, the user (or caregiver) is obliged to familiarize himself/herself with the user manual.

2. INDICATIONS AND CONTRINDICATIONS FOR USE

These medical devices are intended for people with orthopedic, neurological and cardiological diseases and for people after surgery, with musculoskeletal dysfunctions, who have not acquired the ability to sit independently or who have difficulties in adopting and maintaining the sitting position.

Both for flaccid people with reduced muscle tone and for people with spasticity. The device can also be really useful for people who require long-term relief of one or both lower limbs. It is an irreplaceable aid in the rehabilitation process, both in terms of movement and social integration. The device is great help during everyday activities such as rehabilitation but also during playing, studying or having meals. The selection of the appropriate size of the device depends on medical indications and anatomical parameters, regardless of the age of the user.

The device is recommended for people in the following cases:

- Cerebral palsy – various types and forms,
- Brain and spinal cord injuries with lower limbs paralysis and weakness, balance and motor coordination disorders with different etiology,
- Demyelinating disease (e.g. multiple sclerosis - SM),
- Meningocele,
- Muscles diseases with weakness and paralysis (dystrophy, myasthenia gravis, myositis),
- Genetic disorders,
- Degeneration and neurological diseases,
- Other condition connected with muscles weakness, paralysis and musculoskeletal disorders,
- After head and spinal cord injuries and spine trauma with paralysis and paresis.

Contraindications for sitting:

- Critical condition of the user.
- Paralysis,
- Deep sedation,
- Unstable fractures with specific orthopedic recommendations, including spine fractures,
- Untreated deep vein thrombosis,
- Hemodynamic instability,
- Identified brain dysfunction when sitting upright,
- Severe arthritis of the cervical spine,
- Pulmonary embolism,
- Recent myocardial infarction in the last 24 hours,
- Fresh postoperative wounds or non-healing wounds in the back area, pelvic girdle area, hip joints and thighs,
- Users with mental disorders should not be left in the device without the supervision of a caregiver.





3. GENERAL CHARACTERISTIC OF THE STROLLER

3.1 STANDARD EQUIPMENT

- ① Foldable aluminium frame equipped with a front wheel disc brake and rear wheel center brake
- ② Ergonomic push handle
- ③ Backrest with adjustable tilt angle and a storage pocket
- ④ 5-point safety belts with a magnetic fastener
- ⑤ Seat cushion with VISmemo™ foam
- ⑥ Adjustable footrest
- ⑦ Removable inflatable rear wheels
- ⑧ Removable inflatable front wheels
- ⑨ Waterproof upholstery HYDRO™
- ⑩ Special lock of the stroller against accidental unfolding with magnetic fastener





3.2 ACCESSORIES



KKN_001



LED lights



KKN_134
KKN_137



Narrowing pads 6 cm / 10 cm



KKN_147



5-points feet stabilizing belts



KKN_160



Octagon™ headrest



KKN_161



Foot and ankle stabilizing sandals



KKN_162



Dynamic ankle stabilizers (FP-22)



KKN_405



Folding canopy



KKN_408



Rain cover



KKN_409



Mosquito net



KKN_411



Cup holder



KKN_415



Wheels cover



KKN_417
KKN_445



Winter footmuff with muffs
Footmuff



KKN_447



Winter muffs



KKN_452



Phone holder



KKN_501



Bag





 KKN_505

Under seat storage basket



 KKN_506

Stroller cover'n carry bag



 KKN_509

Organizer



 KKN_626

EVO module




 KKN_630

Bicycle tow bar with a flag





4. SIZE TABLE

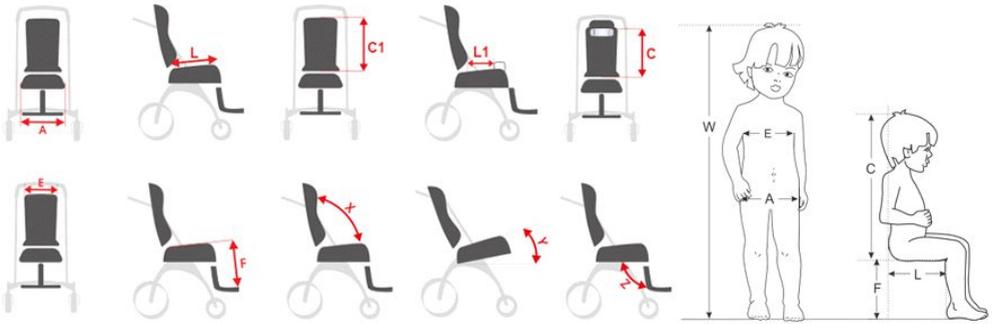

Symbol	Measurement	Unit	Size 1
W	indicative user height*	[cm]	100 - 135
A	seat width	[cm]	37
L	seat depth	[cm]	37
C ₁	backrest height	[cm]	57
C	headrest height	[cm]	47 - 60
E	backrest width	[cm]	36
F	footrest length	[cm]	27, 40
	push handle height	[cm]	82 - 120
X	backrest tilt angle	[°]	100 - 120
Y	seat tilt angle	[°]	20
Z	footrest tilt angle	[°]	125
	maximum user weight	[kg]	40

*The value cannot be main factor for the selection of product size.

WEIGHT AND DIMENSIONS

Measurement		Size 1
Width	[cm]	67
Width after folding**	[cm]	54
Length (min - max)	[cm]	124 - 143
Length after folding**	[cm]	97
Height	[cm]	105
Height after folding**	[cm]	38
Weight	[kg]	14

NOTE: Measurement tolerance range is +/- 1 cm / 1/2 in.

NOT E: Approximate values, depending on the product configuration.

** Folded dimensions are for a stroller with removed wheels and an unfastened footrest.





5. HOW TO USE THE STROLLER

KEY



Info icon - appears in the manual when you need to read the additional information.



Info icon - appears in the manual when you need to press the button shown in the photo



Info icon - appears in the manual when you need to use the tools indicated in the assembly description.



Warning icon - appears in the manual when you need to take special precautions.



Incorrect product adjustment can lead to such side effects as fainting, body deformities, joint overload, etc. If the abovementioned symptoms occur, stop using the product immediately, take the user out of the product and lay them down. If the undesirable symptoms persist, contact a physician immediately. If the symptoms have disappeared, inform the attending physician about this fact in order to make a decision about the further rehabilitation process.





5.1 UNFOLDING THE STROLER



Keep the user away from the moving parts of the stroller while unfolding it.
There is a serious risk of injury.



Pay particular attention to foldable parts of the stroller frame, which pose a risk of injury.

There shall be no upholstery between the moving parts of the stroller.

Before using the stroller, check if all elements are properly locked.



5.2 FOLDING THE STROLLER



Pay particular attention to foldable parts of the stroller frame, which pose a risk of injury.

There shall be no upholstery between the moving parts of the stroller.





5.3 FOOTREST

i

To change the height of the footrest, press the levers located on both sides of the footrest. It is possible to set the height of the footrest on three levels.

Remember to secure the footrest cover by fastening the belt behind the footrest construction (fig. 14 - 15).





5.4 REMOVING THE WHEELS

FRONT WHEEL



REAR WHEELS





5.5 5-POINTS SAFETY BELTS

i

The belts can be mounted at a desired height on the backrest of the stroller - remember to mount both upper straps of the 5-point safety belts at the same height. Each time before using the stroller, fasten the 5-point safety belts and check if they are properly fastened.







i

The upper straps are fastened behind the backrest of the stroller.

After opening the backrest pocket, secure the upper straps with the nuts.



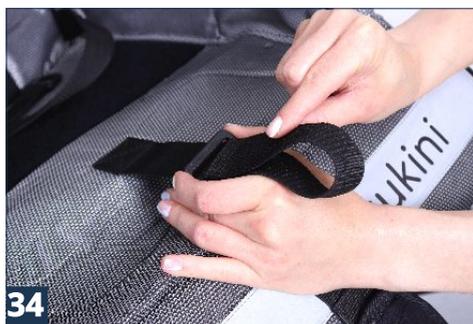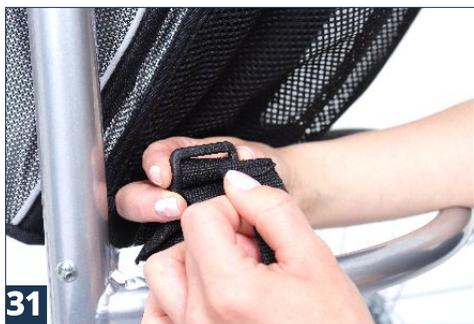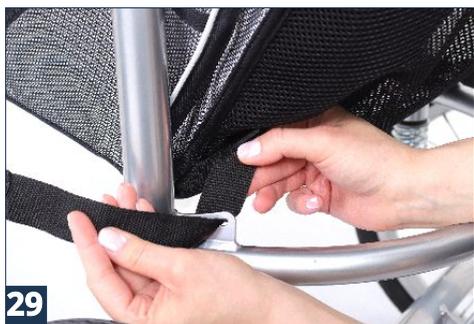

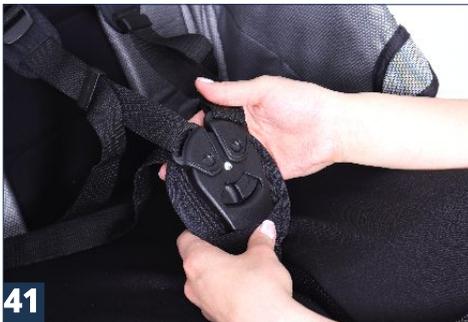

UNLOCKED THE 5-POINTS SAFETY BELTS



5.6 ADJUSTABLE BACKREST



Before using the stroller, check if the locking element of the backrest angle is properly secured.

Maximum load of the storage pocket behind the backrest is 1 kg!

5.7 PUSH HANDLE ADJUSTMENT



5.8 CENTRAL BRAKE / HAND BRAKE



49 WHEELS UNBLOCKED



50 WHEELS BLOCKED



51



You can lock the hand brake by pressing the locking mechanism (fig. 51).

Before use, check if the brakes are working properly - the stroller should be loaded.

5.9 ADJUSTMENT OF THE SPRINGS



It is possible to change the stiffness of the stroller springs, which allows for stiffening the construction or increasing its shock absorption. Adjustment should be made with a wrench included in the set.



52



53



6. ACCESSORIES

6.1 KKN_134 NARROWING PADS 6 CM KKN_137 NARROWING PADS 10 CM



To mount narrowing pads use velcro straps under the seat cushion.





6.2 KKN_147 5-POINTS FEET STABILIZING BELTS



59



60



61



62



63



64



65






66

spanner 8
hex key 3



6.3 KKN_160 OCTAGON™ HEADREST

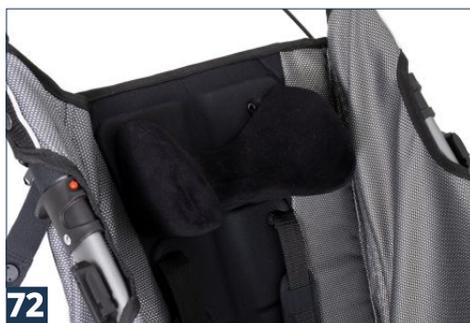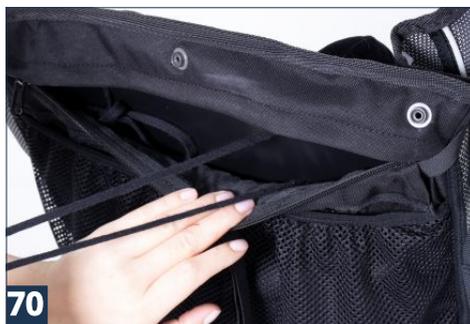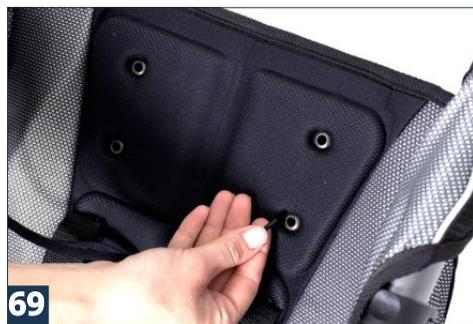



6.4 KKN_161 FOOT AND ANKLE STABILIZING BELTS / KKN_162 DYNAMIC ANKLE STABILIZERS (FP-22)



73



74



Make sure the straps of the sandals stabilizing feet and ankles are not too long - otherwise they pose a risk of getting caught in the front wheel of the stroller.

Adjust the length of the straps by cutting them so that they do not hang under the footrest. Also remember to adjust the length of the straps so that the sandals are securely fastened to the footrest.



75



76



77




78










6.5 KKN_405 FOLDING CANOPY

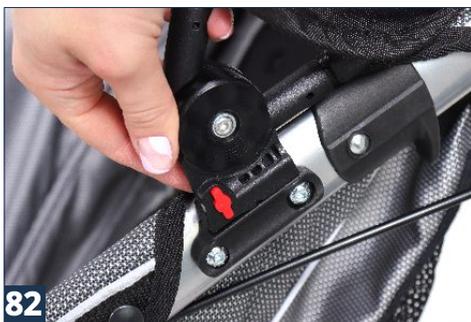

i

In order to remove the folding canopy, press the buttons on both sides of the mounting elements on the frame (**fig. 83**) and pull it out.

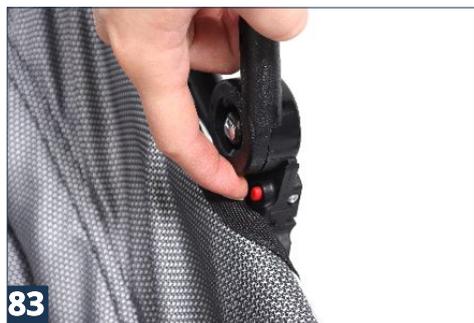

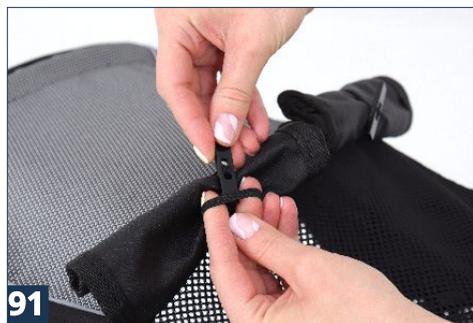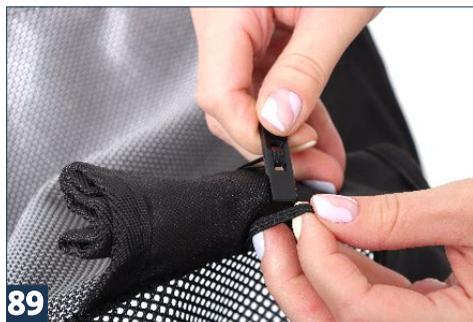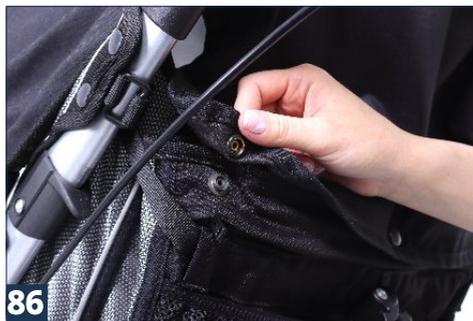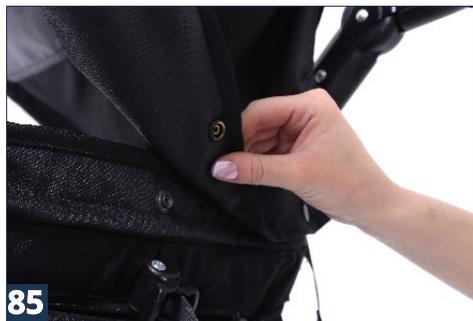



6.6 KKN_408 RAIN COVER KKN_409 MOSQUITO NET



Rain cover can be adjusted to the different heights of the footrest.







6.7 KKN_411 CUP HOLDER / KKN_452 PHONE HOLDER



6.8 KKN_415 WHEELS COVER





6.9 KKN_445 FOOTMUFF



6.10 KKN_505 UNDER SEAT STORAGE BASKET





Maximum load of the under seat storage basket is 3 kg!

6.11 KKN_506 STROLLER COVER 'N CARRY BAG





125



126



127



128

6.12 KKN_630 BICYCLE TOW BAR WITH A FLAG



Mount the tow bar linkage to the bicycle as described in the instructions provided with the kit.



129



130





Before mounting the bicycle tow bar, remove the front wheel of the stroller (see page 14, pt. 5.4 Removing the wheels, fig. 18 - 19).



131



132



133



134




135



136



137

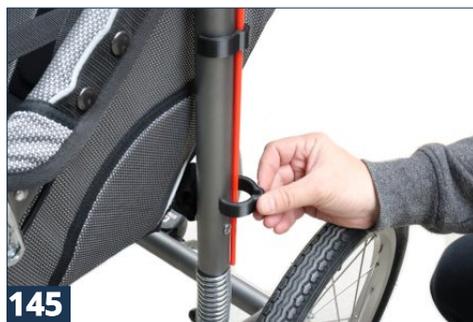

6.13 KKN_626 EVO MODULE

i

Before mounting the EVO module, remove the front wheel of the stroller (see page 14, pt. 5.4 Removing the wheels - front wheel, fig. 18 - 19).



7. THE USE OF STROLLER



KUKINI™ stroller has to be used with compliance to guidelines included in this user manual. Remember to use the product only with someone who is trained and knowledgeable.

Do not forget to prepare both product as well as user before using stroller!

7.1 PREPARATION OF THE USER

Before using the product, it is essential to make sure that:

- user's outfit is light and comfortable sporty clothes are advisable;
- shoes should be comfortable and, moreover, they should stabilize ankle;
- due to hygienic reason and to avoid abrasion it is advisable for the patient to always wear clothes when using the device, skin must not be naked especially in places of regular contact with upholstery;
- drains, bags, catheters etc. if applied, have to be secured to function properly and not to disturb during using the product;
- if it is necessary for a user to wear orthopedic supply (collar, corset, orthesis, shoes) if it is recommended by doctor.



Outfit should protect user's skin from direct contact with upholstery as well as other parts of equipment.

Only doctor can make a decision concerning using any kind of orthopedic supply!

7.2 PREPARATION OF THE STROLLER

1. Before use it is necessary to:

- unfold the device;
- place the device on even ground;
- make sure that all parts are serviceable, correctly mounted and stable;
- block brakes;
- adjust the device to user's measurements;
- undo safety belts so as to place easily the user in the device;
- place user in the device;
- fasten all elements which stabilize user e.g. safety belts;
- mount additional equipment e.g. foot and ankle stabilizing sandals.

2. In case of patients who have problem with bladder, it is necessary to put special cloth on seat. This cloth can be purchased in pharmacy or medical shops.



In case of any obstacles such as step or curb, it is necessary to raise front wheels by pressing the handle down . Next, place front wheels on step and then raise back part of stroller with the aid of handle.

Be careful not to tilt the stroller back too much.

Remember to place stroller forwards.

It is essential to hold stroller with the aid of handle during using it otherwise stroller can roll.

Before the product is used, it is highly necessary to make sure that all belts as well as safety equipment are correctly fastened as well as adjusted. What is more, check if they do not become undone due to the pressure and, moreover, make sure that clampings from adjustable elements are firmly tightened!

Equipment works correctly only when the device is adjusted according to user's height as well as weight. Remember to take all these measurements into account while purchasing the product. Correct adjustment is the most important due to the fact that only then the product is safe. It is forbidden to force child to keep specific position because he/she will feel uncomfortable and it may lead to wrong posture.

8. SAFETY RULES



KUKINI™ stroller has to be used in compliance with guidelines in this user manual.

It is highly important to consult either doctor or physiotherapist in order to make right decision concerning the choice of stroller and its size. It is also essential to familiarize with the guidelines described in user manual before using the device.

1. Before use it is necessary to:

- make sure that there are no damages or different faults which could endanger user's life; check if all parts are working and are correctly mounted and, moreover, check if they are not broken,
- check if all screws, knobs and all glued, sewed or tapped parts are tightened,
- check if brakes are operating correctly,
- check if buckles are operating correctly,
- always use safety belts, velcro tapes should overlap min. 8cm,
- adjust the device according to user's measurements and needs e.g. the adjustment of pelottes, footplate etc.,
- adjust the device according to user's measurements and needs e.g. the adjustment of footplate etc.,
- secure user with the aid of elements such as foot stabilizing belts, headrest etc. It is highly important to secure user because it safeguards him/her falling out,
- check if the wheels are correctly mounted and inflated, if there is not enough air, the wheels must be inflated to a maximum pressure of 2,4 bar.

2. It is forbidden to use stroller for different purposes that it is intended to.

3. It is forbidden to use stroller when it is damaged or any part is missing.

4. It is forbidden to leave the user unattended.

5. Stroller should be used only when there is someone who is trained and knowledgeable.

6. It is forbidden to leave stroller and the user on unstable ground unattended, even if the brake is engaged.

7. It is forbidden to either go up or down the stairs; it doesn't matter if the user is placed in the device or not.

8. It is important to engage brake during getting into and getting out the stroller.

9. Make sure that the user does not stand on the footrest when getting in and out of the stroller.

10. Make sure that all limbs are not close to spokes or other adjustable elements.
11. The stroller should be stored and used away from sources of fire or high temperature. Moreover, stroller shouldn't be stored in rooms with high humidity.
12. Secure stroller during transportation in order to avoid any possible damages. Manufacturer doesn't take responsibility for damages which appeared due to improper preparation of the device during transportation.
13. Before using the stroller, remove all paper and plastic packaging. The packaging is not a toy! Keep away from children and animals.



Incorrect product adjustment can lead to such side effects as fainting, body deformities, joint overload, etc. If the abovementioned symptoms occur, stop using the product immediately, take the user out of the product and lay them down. If the undesirable symptoms persist, contact a physician immediately. If the symptoms have disappeared, inform the attending physician about this fact in order to make a decision about the further rehabilitation process.

Serious incidents involving this device should be reported both to manufacturer and to the competent authority in your country.

It is forbidden to place the user in the device while being in car, bus or train.

The product contains small parts which children may swallow.

Do not use a product that has any signs of damage, tearing or wear. Only an undamaged product and proper use guarantee its right and safe working. If the product is damaged, discontinue use and contact the manufacturer.

Under no circumstances should an unqualified person repair or alter the product and its construction, as the health and safety of the user depend on it as well as it will lead to the loss of warranty.

Remember that all the loads hung on the push handle may cause the stroller to lose its balance.

Maximum load of the storage pocket behind the backrest is 1 kg!

Maximum load of the under seat storage basket is 3 kg!

The weight of user should not be bigger than maximal weight described on page 9 where you will find information regarding this issue.

9. TERMS OF WARRANTY

- 1) **Akces-MED Ltd.** sells products free from manufacturing defects or faults
- 2) Manufacturer gives the purchaser 24-months guarantee from the date of purchase. All defects revealed during warranty period will be repaired free of charge.
- 3) All defects which appear after warranty period can be repaired only when the user will pay for it. This rule doesn't apply to refunded products - user can ask National Health Refund for refund.
- 4) Manufacturer claims that the product can be used for 48-months. User can use this product after this period only when he/she will send the device to manufacturer so as to service it. Next, manufacturer determines again how long the product can be used and, moreover, the date of the next service will be determined.
- 5) If product is damaged, please contact our dealer or **Akces-MED Ltd.** customer service (ph. +48 17 864 04 77, e-mail: export@akces-med.com).

10. MAINTENANCE AND CLEANING

The stroller **KUKINI™**, which is a medical device, should always be kept clean and should be used in accordance with the manufacturer's instructions. The stroller should be kept in a clean, dry and ventilated room. Maintenance and cleaning must be done at least once a month. Periodic cleaning and maintenance of the chair is essential to keep its proper functioning and safety.

Recommendations for cleaning the upholstery and other elements of the stroller:

1. Wash with warm water using a soft cloth and mild cleaners.
2. Make sure the cloth is not too wet, as this may enlarge the stain and damage the upholstery or foam inside the upholstery
3. Remember to remove excess moisture from the upholstery before the next cleaning.
4. After cleaning the stains, the upholstery should be rinsed with lukewarm water and dried with a clean cloth.
5. You must not soak the upholstery.
6. The upholstery must not get wet during cleaning
7. Do not use hard brushes for cleaning the upholstery
8. Do not clean the upholstery with strong chemicals, e.g. solvents or chlorine-based caustics.
9. Do not clean, replace any elements or maintain the product during use.
10. Metal and plastic elements should be cleaned by hand with suitable cleaners, after having read the recommendations of the manufacturer of the given cleaner.

11. It is possible to disinfect metal, plastic, wooden and upholstery elements of the product with a generally available disinfectant. In order to disinfect the selected element, spray its surface with a disinfectant generally available on the market and wipe with a dry, clean cloth. After disinfection, wait until the element is completely dry, especially in the case of upholstery. Before using the disinfectant, first check in an invisible place whether the product will not discolor or damage the disinfected surface.



Remember that upholstery has to be always dry before using!

The manufacturer of the stroller, Akces-MED Ltd., is not responsible for any damages caused by using unsuitable cleaners.

Before using it is necessary to:

- clean wheels,
- check if base as well as wheels are correctly mounted; check if brake stops turning wheel,
- check if wheelchair construction is free of breaks or any kinds of deformities,
- check if all movable parts are working and all screws, nuts, knobs etc. are correctly tightened,
- check if belts, vests, trunk belts, feet stabilizing belts etc are not frayed; check if buckles and clamps work correctly.



Manufacturer takes responsibility only for hidden faults or damages which appeared despite the fact that user complied with the guidelines.

In case of any damages, it is essential to contact the person who sold you the product.





TECHNICAL INSPECTION

Technical inspections should be performed regularly. Before servicing, the product should be thoroughly cleaned to reveal any hidden damage.

Frequency	What should be checked? What should be done?	Remarks
Before each use.	<p>General condition of the product.</p> <p>Check:</p> <ul style="list-style-type: none"> the entire product for any visible damage, cracks or mechanical defects,, if the product complete, if all bolts, screws, knobs, nuts and other elements of the construction are complete and tightened.. 	In case of any damage or defects, discontinue the use of the product and contact your Supplier/Distributor or the Manufacturer.
	<p>Wheels and brakes.</p> <p>Check:</p> <ul style="list-style-type: none"> if the wheels are properly mounted and if they are not damaged, if the brake is blocking the spinning wheel, if the wheels are not dirty. If necessary, clean the wheels. 	In case of any damage or defects, discontinue the use of the product and contact your Supplier/Distributor or the Manufacturer.
	<p>Fastening of the construction elements.</p> <p>Check all the mechanisms of the product and its moving parts. Check in particular:</p> <ul style="list-style-type: none"> if the seat is properly mounted (if applicable), if the wheels are properly mounted (if applicable), if the elements securing the user are properly mounted and if they work well (if applicable), if all the moving parts are complete, work properly, do not jam, if the adjustment mechanisms of the following elements work properly (if applicable): headrest, backrest, seat, abduction block, armrests, footrests. 	In case of any damage or defects, discontinue the use of the product and contact your Supplier/Distributor or the Manufacturer.
	<p>Upholstery elements.</p> <p>Check:</p> <ul style="list-style-type: none"> if there is any damage, tears, rips, loose seams or deformations on the upholstery elements, if the stabilizing straps, vests or safety belts are not frayed or twisted and if the buckles work properly, if the stabilizing straps, vests or safety belts work correctly and do not unfasten under pressure or when pulled. 	In case of any damage or defects, discontinue the use of the product and contact your Supplier/Distributor or the Manufacturer.
Once a week.	<p>Product cleaning.</p> <p>If the frame and upholstery get dirty (dusty, muddy, etc.) during everyday use, wipe them with a damp cloth without detergents. Remove dirt (e.g. hair, food remains, etc.) from the moving parts of the product.</p>	The product should be cleaned when necessary, but at least once a week. Do not use any chlorine based cleaners or methylated spirit. Before using the product, check if the upholstery is dry.





Frequency	What should be checked? What should be done?	Remarks	
Once a month.	Wheel axles, quick connectors (if applicable).	<p>Check:</p> <ul style="list-style-type: none"> if the wheels are rolling and if they are not loose. To maintain the wheels, apply a small amount of a preservative-lubricant between the axle and the wheel hub. Check if you can easily remove and attach the wheels. 	<p>Clean the wheels when dirty.</p> <p>In case of any damage or defects, discontinue the use of the product and contact your Supplier/Distributor or the Manufacturer.</p>
	Product construction connectors (welding, threading, soldering)	<p>Check:</p> <ul style="list-style-type: none"> the condition of the product frame in the places of joining (welds, threads, soldering). 	<p>In case of any damage or defects, discontinue the use of the product and contact your Supplier/Distributor or the Manufacturer.</p>
	Tires/innertubes (if applicable).	<p>Check:</p> <ul style="list-style-type: none"> tire wear condition. Check for visible damage. Check tire pressure (if applicable). 	<p>If the tire pressure is too low, inflate them to the correct level.</p> <p>In case of any damage or defects, discontinue the use of the product and contact your Supplier/Distributor or the Manufacturer.</p>
Once every 6 months.	Construction moving parts.	<p>Check:</p> <ul style="list-style-type: none"> if the moving parts of the construction have not been damaged. Maintain them using a proper cleaning agent. 	<p>In case of any damage or defects, discontinue the use of the product and contact your Supplier/Distributor or the Manufacturer.</p>
	Product labeling.	<p>Check:</p> <ul style="list-style-type: none"> if the marking and labels are readable. 	<p>In case of any damage or defects, discontinue the use of the product and contact your Supplier/Distributor or the Manufacturer.</p>
After the life time of the product finishes.	Detailed product check.	Check the product.	<p>The manufacturer specifies the life time of the product, which is given in the user manual. After the given period, the user may use the product only if it is sent back to the manufacturer to be checked first. After that, the manufacturer will re-define the life time of the product and the date of the next check.</p>

The list of all spare parts is available for download after selecting a product on www.akces-med.com.




11. LABELLING

11.1 SYMBOLS

- | | | | |
|--|---|---|--|
|  | Declaration that the product meets the essential requirements |  | Do not chlorinate |
|  | Catalog number |  | Do not iron |
|  | Serial number |  | Do not dry in a drum dryer |
|  | Please read the manual instructions before use |  | Do not bleach |
|  | You need to take special precautions |  | Dry vertically |
|  | Manufacturer |  | Careful! Delicate content |
|  | Production date |  | Prohibition to leave the user without care during use of the product |
|  | The product does not contain latex |  | It is forbidden to either go up or down the stairs; It doesn't matter if the user is placed in |
|  | Store in a dry place |  | You need to read the additional information |
|  | Prohibition of use without fastened safety belts |  | You need to press the button shown in the photo |
|  | Protect from sunlight |  | You need to use the tools indicated in the assembly description |
|  | Storage temperature limitation |  | Medical Device |
|  | Relative humidity storage | | |

11.2 SAMPLE DATA PLATE

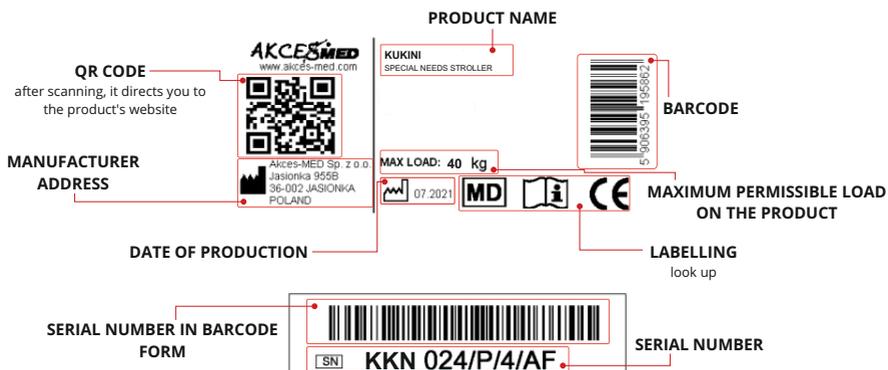

MANUFACTURER



Jasionka 955B
36-002 Jasionka
POLAND ph. +48 17 864 04 77
e-mail: export@akces-med.com
BDO: 000031619



Authorised Dealer



akces-med.com

Date of issue: 14.06.2021
The last update: 04.05.2022
© Sławomir Wroński 2022